



Instant-view<sup>®</sup> plus


# COVID-19 Antigen Test




## Quick Reference Guide

After specimen is collected, complete testing procedure as follows:


- 1** Remove white cap and discard.

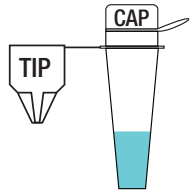


- 2** Insert swab, swirl 3 times and let set for 1 minute.

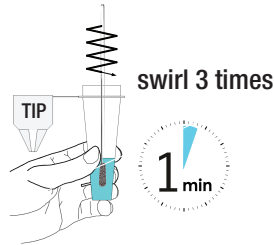


- 3** Squeeze liquid from swab head inside tube while removing swab, close tip and push down to seal.

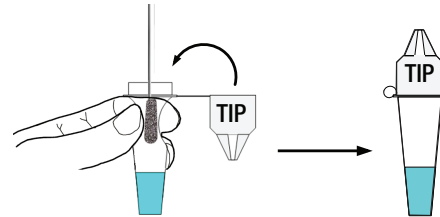

- 4** Remove device from pouch, set on flat surface and add 3 drops into sample well.

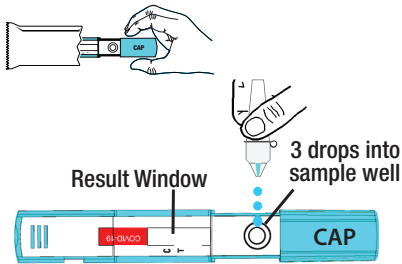


- 5** Wait until the burgundy color flow appears in the Result Window before closing the cap. Push cap closed and set timer between 3-7 minutes.

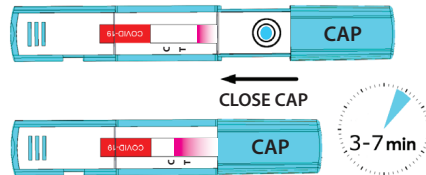

- 6** Do not read results before 3 or after 7 minutes.

**C - Control Line**  
**T - Test Line**

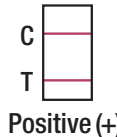

Positive (+)

**Positive Result**  
Two (2) burgundy lines at T and C positions is a positive test result.



Negative (-)

**Negative Result**  
One (1) burgundy line only at the C position is a negative test result.



Invalid

**Invalid Result**  
No line at the C position is an invalid test result.